

SURTEX
INSTRUMENTS

Crafted to Excellence



Usage and fields of application:
MICROSURGERY FOR LUMBAR DISC DISEASE

Special Features:

- Anti-Glare black coating on all parts
- Fixed and hinged retractor frame arms (where required)
- Lumbar microdiscectomy
- Lateral zone decompression
- Paraspinal approach to foraminal HNP
- Laminoplasty for spinal canal stenosis

The problem it solves:

Flexibility to do almost any spine procedure, from advanced microscopic-aided procedures to multi-level open surgery.

Less-invasive approaches to microscopic repair of herniated discs.



*“Convenience and flexibility for the surgeon.
&
Reduced trauma for the patient.”*




Usage and fields of application:

MICROSURGERY FOR LUMBAR DISC DISEASE



- The surgeon positions the blades prior to inserting the retractor frame.
- This allows accurate placement.
- Two frames placed together end to end provide multilevel exposure for routine open spinal surgery.

HOW TO MEASURE BLADES CORRECTLY:

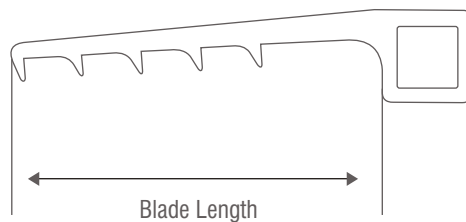



Surtex Basic McCulloch Retractor Set

Qty	Product Codes	Description
1 Pc.	RT-990-43	Retractor Frame
4 Pairs	RT-993-04 up to RT-993-07	Wide Muscle Blades 4 cm to 7 cm long
4 Pairs	RT-994-04 up to RT-994-07	Narrow Muscle Blades 4 cm to 7 cm long
4 Pcs.	RT-991-03 up to RT-991-06	Hook Blades 3 cm to 6 cm long
4 Pcs.	RT-992-03 up to RT-992-06	Multi-Toothed Blades 3 cm to 6 cm long
1 Pc.	ST-482-02	Surticon™ ¾ Instruments Sterilization Basket 405 x 253 x 50 mm


Surtex Advance McCulloch Retractor Set

Qty	Product Codes	Description
2 Pcs.	RT-990-43	Retractor Frame
6 Pairs	RT-993-03 up to RT-993-08	Wide Muscle Blades 3 cm to 8 cm long
6 Pairs	RT-994-03 up to RT-994-08	Narrow Muscle Blades 3 cm to 8 cm long
6 Pcs.	RT-991-02 up to RT-991-07	Hook Blades 2 cm to 7 cm long
6 Pcs.	RT-992-02 up to RT-992-07	Multi-Toothed Blades 2 cm to 7 cm long
1 Pc.	ST-482-02	Surticon™ ¾ Instruments Sterilization Basket 405 x 253 x 50 mm

Retractor Frame, Rigid Arms

Standard frames are blackened to reduce glare


Catalog #	Profile
RT-990-41	Max. Spread 70 mm
RT-990-42	Max. Spread 90 mm

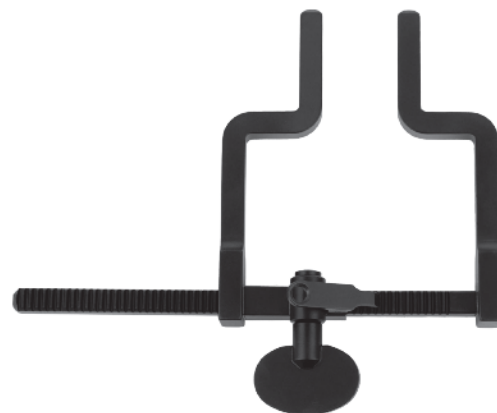

Retractor Frame, Jointed Arms

The hinged frame lies flat on the patient's back:
minimal obstruction to other instruments

Catalog #	Profile
RT-990-43	Max. Spread 70 mm
RT-990-44	Max. Spread 90 mm
RT-990-45	Max. Spread 150 mm

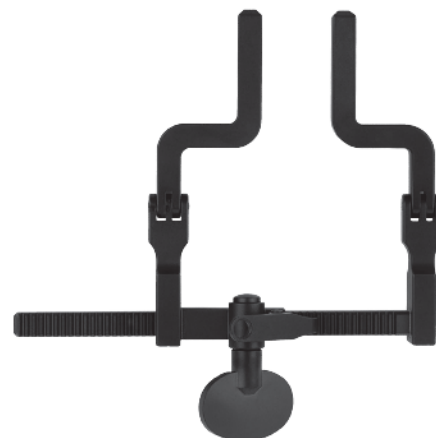



Hook Blade

For engaging the spinous process

Catalog #	Length
RT-991-02	2 cm
RT-991-03	3 cm
RT-991-04	4 cm
RT-991-05	5 cm
RT-991-06	6 cm
RT-991-07	7 cm
RT-991-08	8 cm
RT-991-09	9 cm
RT-991-10	10 cm
RT-991-18	18 cm



Multiple-Toothed Blade

For hemi-laminectomy procedures

Catalog #	Length
RT-992-02	2 cm
RT-992-03	3 cm
RT-992-04	4 cm
RT-992-05	5 cm
RT-992-06	6 cm
RT-992-07	7 cm
RT-992-08	8 cm
RT-992-09	9 cm
RT-992-10	10 cm
RT-992-18	18 cm



Wide Muscle Blade (Width 27 mm)

For full microsurgical exposure

Catalog #	Length
RT-993-03	3 cm
RT-993-04	4 cm
RT-993-05	5 cm
RT-993-06	6 cm
RT-993-07	7 cm
RT-993-08	8 cm
RT-993-09	9 cm
RT-993-10	10 cm
RT-993-11	11 cm
RT-993-12	12 cm
RT-993-18	18 cm



Narrow Muscle Blade (Width 20 mm)

For small incision sites

Catalog #	Length
RT-994-03	3 cm
RT-994-04	4 cm
RT-994-05	5 cm
RT-994-06	6 cm
RT-994-07	7 cm
RT-994-08	8 cm
RT-994-09	9 cm
RT-994-10	10 cm
RT-994-11	11 cm
RT-994-12	12 cm
RT-994-18	18 cm



Thin Muscle Blade (Width 17 mm)

Catalog #	Length
RT-995-03	3 cm
RT-995-04	4 cm
RT-995-05	5 cm
RT-995-06	6 cm
RT-995-07	7 cm
RT-995-08	8 cm
RT-995-09	9 cm



Extra Thin Blade (Width 10 mm)

Catalog #	Length
RT-996-03	3 cm
RT-996-04	4 cm
RT-996-05	5 cm
RT-996-06	6 cm
RT-996-07	7 cm
RT-996-08	8 cm



Cervical Blade (Width 20 mm)

For small incision sites providing full micro surgical exposure

Catalog #	Length
RT-997-04	4 cm
RT-997-05	5 cm
RT-997-06	6 cm
RT-997-07	7 cm



COPYRIGHT NOTICE

The copyright, trademarks, domain names and other intellectual property rights in all materials and information in this presentation belongs to Surtex Instruments Ltd. All rights are reserved. Technical alterations are possible. This presentation may be used for no other purpose than offering, buying and selling of Surtex Instruments products. No part may be reproduced or copied in any form. In the event of misuse we retain the right to recall our catalogues and take appropriate legal action.

DISCLAIMER

This documents provides general information about Surtex Instruments Ltd, its products and services, and summarises general capabilities and offerings, which we deem to be relevant to our customers. Whilst Surtex Instruments has taken appropriate steps to ensure the accuracy of the information contained in the document Surtex Instruments gives no warranty regarding the accuracy or completeness of such information.

The presentation was prepared to provide information and does not constitute a contract. Surtex reserves the right, without prior notice, to change, delete, supplement, or otherwise amend at any time the information, images and offerings contained in the presentation or other documents. Surtex shall not be liable for any loss or damage of whatever nature (whether direct, indirect, consequential or other) may arise as a result of any third party relying on the information contained in this document.

For further detailed information, prices, terms and conditions, customer should contact Surtex Instruments at sales@surtex-instruments.co.uk.



47 Kingston Hill, London
KT2 7PS, United Kingdom
Phone: +44 (0) 208 123 3513

E-mail: sales@surtex-instruments.co.uk
Website: www.surtex-instruments.co.uk

